

OPTIMA

The only electric handpiece that can be used at home and in the clinics

CAMPUS
EDITION



Train at home!

Optima works with and without compressed air and water. Can be used at home by simply plugging into a standard outlet. Connect to a dental unit to use in clinic with water spray.



Optima Campus System without MX2 Plus motor REF 1600959-001E
MX2 Plus motor REF 1600677-001

OPTIMA CAMPUS



Using at home without air and water


STEP 1

Connect the power cable from the transformer



STEP 2

Flip the Switch to position I to use at home without air

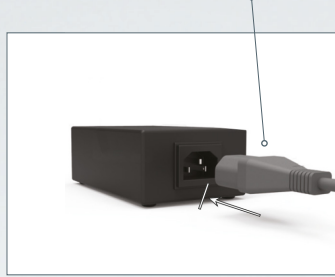

STEP 3

Connect the power to a wall outlet



STEP 4

Connect the tubing to the micromotor

STEP 5


Control speed with the dial



Connect to any dental unit and use on patients


STEP 1

Connect the power cable from the transformer

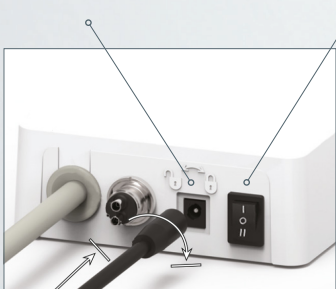

STEP 2

Flip the Switch to position II when connected to a Unit

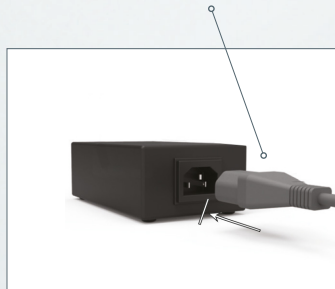

STEP 3


Connect the power to a wall outlet

STEP 4

Connect the tubing to the micromotor

STEP 5

Control the speed with the foot control

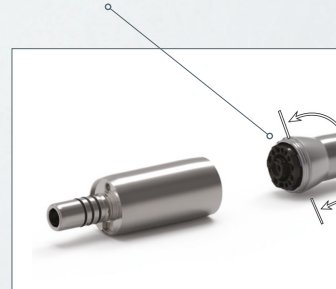

STEP 6

Connect a standard 4-hole tubing from the dental unit



TO ORDER CALL 800 433 2436 OR CONTACT YOUR LOCAL REP

Southern CA (ZIP 90000-93499)
HI
AMY MCDONALD
(562) 393-0686
amy.mcdonald@bienair.com

Northern CA (zip codes 935+)
Northern NV
MAX MURPHY
(510) 326-8592
max@murphydentalsales.com

OR, WA, AK, ID
CAMERON CONDRAV
(503) 347-9035
nwdentalsupply@gmail.com

CO, UT, MT, WY
MIKE BORER
(303) 981-5281
mike@rmsadental.com

MN, NE, ND, SD, IA, Northwest IA
TIM VAN WYCHEN
(651) 231-2801
vanwychen@hotmail.com

KS, MO, WI, IL
KARA SKEAN
(847) 513-2329
kara.skean@bienair.com

MI, IN, OH, KY
GUY GISMONDI
(734) 516-1597
guy@kagedist.com

AZ, NM, El Paso
Southern NV
OPEN TERRITORY
(800) 433-2436
sales-us@bienair.com

TX, OK, LA
LEAH N. GARCIA
(214) 650-5956
leah.garcia@bienair.com

WV, NY, PA
MIKE MATUCH
(716) 913-8534
michaelmatuch@yahoo.com

MA, RI, NH, ME, VT, CT
PAUL SNELL
(978) 852-5229
pauls@prrdental.com

NY (zip codes 1000-12000),
Northern NJ (zip codes 070-089)
KRISTIN HANNA
(631) 241-6091
kristin.hanna@bienair.com

Eastern PA,
Southern NJ (zip codes 089+)
MARK FRANKENFIELD
(484) 849-1065
mtf576@msn.com

MD, VA, DC, DE
GRETCHEN KUNEY
(703) 727-4483
gretchen.kuney@bienair.com

AR, MS
OPEN TERRITORY
(800) 433-2436
ba-us@bienair.com

GA, AL, FL (ZIP 323-325)
NATHAN SHAW
(678) 778-1295
snate1975@aol.com

TN, NC, SC
CARTER HULL
(704) 999-5032
carter.hull@bienair.com

FL
CHRIS COLLINS
(407) 435-5150
chris.collins@bienair.com

CANADA

QC, NF, NB, NS, BS, PQ, ON,
Manitoba & Saskatchewan CAN
DAVID GOLDSHAW
(647) 298-7429
aushaw@rogers.com