

# HOW TO PREVENT PREMATURE

## HANDPIECE FAILURE

SIMPLE ADVICE TO EXTEND YOUR HANDPIECES' LIFE!

### HANDLE WITH CARE

01

- **Avoid Drops & Impact** – Always place handpieces securely in their holders to prevent falls.
- **Store Smartly** – Use padded trays or holders instead of loose storage in drawers or on hard surfaces.
- **Be Gentle with Burs** – Always insert and remove burs by hand.



02

### KEEP IT CLEAN & COOL

- **Less Is More and More is Less** – Use the proper amount of Spraynet® and Lubrifluid® to prevent excess friction and wear.
- **Clean Before You Lube** – Ensure old lubricant is properly cleaned out before reapplying to prevent buildup.
- **Cool Down First** – Let handpieces cool for at least 90 minutes after autoclaving before use to avoid overheating internal components.



### BURS NOT BUR-DENS

03

- **Remove Burs Before Sterilization** – Leaving them in can cause damage to the chuck.
- **Fully Insert for Full Performance** – Never extend a bur beyond its intended position, as this can cause immediate damage and/or injure the patient.
- **Inspect Before You Cut** – Check burs regularly for bends or wear, especially when cutting tough materials like zirconia.



Learn more at [Bienairacademy.com](https://www.bienairacademy.com)