

# Contra-angle Reprocessing

## Cleaning, Lubrication, And Sterilization

### STEP 1

Remove the bur



### STEP 2

Clean external surfaces

Clean the external surface using a clean, soft-bristled brush and non-linting wipe.

**Do not** use corrosive detergents that contain chlorine, acetone aldehydes, or bleaches.

**Do not** soak attachment (e.g.: ultrasonic cleaners, saline solutions).

NOTE: If you have a Lubricare 2 Maintenance System, you can skip steps 3 & 4 as it cleans and lubricates attachments automatically.

### STEP 3

Clean with Spraynet<sup>®</sup>  
REF 1600036-006

Spray for 1 second



### STEP 4

Lubricate with Lubrifluid<sup>®</sup>  
REF 1600064-006

Spray for 1 second



### STEP 5

**Sterilization:** Allow the attachments to complete a full sterilization cycle before use, following the manufacturer's IFU.

Tips for Proper Sterilization:

Always remove the bur.

Always place the sterilization bag paper side up.

Always allow attachments to cool to ambient temperature before use.

S W I S S  M A D E