


# LUBRICARE 2

## STEP-BY-STEP GUIDE

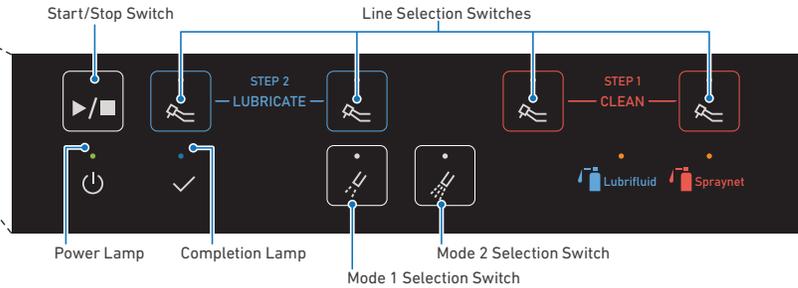


### STEP 1: CLEAN WITH **SPRAYNET**




Check that the Completion Lamp is lit at the end of each cycle before opening the front door.

- Insert up to two attachments to the couplers on the **RIGHT**, with the chucks facing inside.
- Press the “Clean” buttons, then press the Start/Stop Switch.
- After cleaning, press and hold the release button then pull down to remove the attachments.

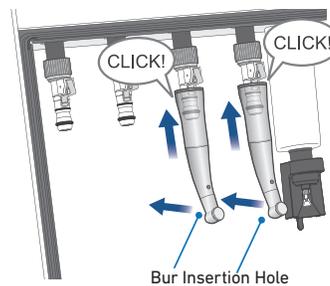


Listen for the click when connecting to the coupler.

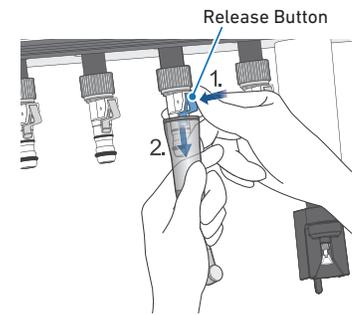


Hold the Release Button, then pull attachment down to remove.

### STEP 2: LUBRICATE WITH **LUBRIFLUID**

- Insert the cleaned attachments to the couplers on the **LEFT**, with the chucks facing inside.
- Press the “Lubricate” buttons, then press the Start/Stop Switch.
- After lubrication, press and hold the release button then pull down to remove the attachments.

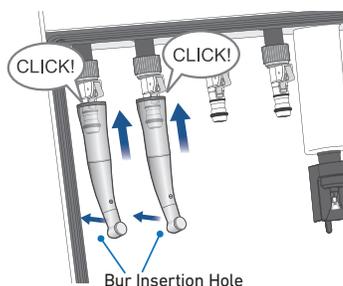

Listen for the click when connecting to the coupler.

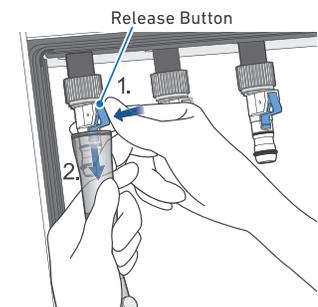

Hold the Release Button, then pull attachment down to remove.

S W I S S  M A D E