

# Using the OPTIMA<sup>INT</sup>



## Programmable Procedures

Program up to 20 procedures/attachments with custom names, ratios, speed, and torque.

Click the and arrows to scroll through them.

## Cursor

The cursor highlights what is or can be selected (speed, setting, etc.).

## Touchpad Availability

The arrow icons identify which way you can go with the cursor.

In this example, you can move the cursor right or down.

## REV = Reverse

Reverses the rotation of the bur or file.

## ESC = Back


Escape will take you back to the previous screen.

## SETUP = Settings

Change light intensity, sound, and other settings.

## SAVE = Keep New Settings

Press save if/when you make an adjustment in the setup menu.

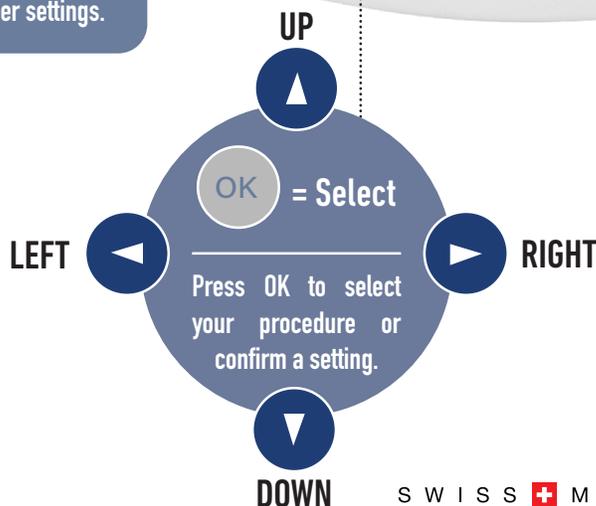

SWISS MADE

For videos and more:



[bienairacademy.com](http://bienairacademy.com)